

BAGUETTE

ANESTHESIA

BAG732 plus



VAROB3



BAGUETTE

ANESTHESIA

BAG732 plus

Anesthesia system supply

Technical data

Volume control, pressure control, electronic PEEP.

Adult-Pediatric-Neonatal range without bellow exchange.

Automatic functioning check (auto-check-system).

Compliance Compensation system (breath by breath).

Alarms for all the ventilator parameters and intelligent alarms for failures with suggestions on screen for problem solution.

B3 vaporizers, sevoflurane 8%, isoflurane 6%, enflurane 5%, halothane 6%.

Fresh gases flow disconnection: Mechanism that supplies fresh air flow to the respiratory circuit only during the expiratory fase, avoiding to add an extra volume into the VT supplied by the ventilator and selected by the user.

Compliance compensation: Mechanism which supplies an additional flowing volume equal to the volume trapped in the circuit that does not get to the patient, due to the distention, caused by the positive pressure, of the circuit components.

Hypoxic Guard System: Allows the user to program the minimal O₂ concentration and cuts off the N₂O and the medical gases when this parameter is passed.

Measuring gas unit: One or two flowmeter tubes for O₂ and N₂O. One flowmeter tube for Air. Gas mixture selection: O₂ and Air or O₂ and N₂O.

Anesthesia Vaporizer

Technical data

VAPOB3

| | | | |
|---------------------|--|---|-------------------------------|
| CONCENTRATION RANGE | Halotane | 0.5 to 5/6% | |
| | Enflurane | 0.5 to 5% | |
| | Isoflurane | 0.5 to 5/6% | |
| | Sevoflurane | 0.5 to 5/7/8% | |
| TEMPERATURE RANGE | Ambient and Vapor temperature | during operation | 10 to 40°C 18 to 30°C |
| | | during shut-down (filled, control dial) | 0 to 40°C |
| | | during storage (empty, dry wick) | -20°C to 70°C -20°C to 60°C |
| FLOW RANGE | 0.2 to 15 L/min | For concentrations 5 vol% | |
| | 0.2 to 10 L/min | | |
| STANDAR CONECTOR | 23 mm Conical conector (ISO 5356-1) and filter | | |

➔ Ask about OEM acquisition



ISO 9001
ISO 13485



CE
ANMAT PM1167-6
ANMAT BPF



IRAM-FAAAR AB 37202

IEC 60601

ISO 21647

UNE-EN 60601-1-2

COMPATIBILIDAD ELECTROMAGNETICA



Av. Rivadavia 10148 / 50 / 52. CABA

(C1408AAO) República Argentina

Tel/Fax: (+54 11) 4635 5885

4682 1241 / 9647 / 5385

www.baguette.com.ar

JOSÉ BAGUETTE E HIJOS S.A.I.C. main activity is to produce anesthesia equipment and accessories, specially low, mid and high complexity machines. We manufacture exclusively this kind of equipment, because we are mainly interested in maintaining a highly reliable line of products.